



Assessing C-Spine with the “XCollar” in Place

Stabilize the front of the XCollar during the following steps:


① Detach Blue & Yellow XStraps

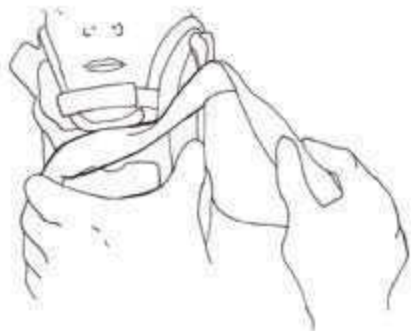


② Loosen the Buckle Side Strap

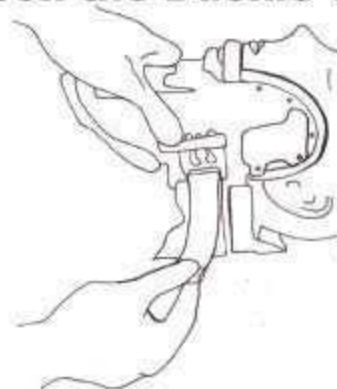

③ Slide hand between C-Spine area & XCollar as needed to assess patient

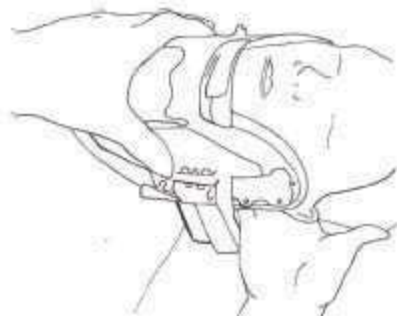

④ Removal: apply even pressure on both sides of the buckle to release

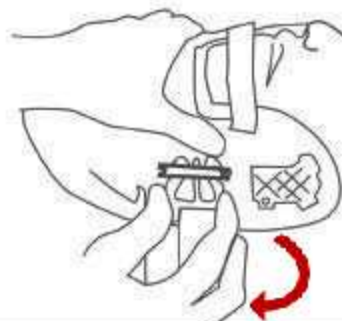

To Re-Apply: use “Opposing Forces” to tighten side strap and secure XStraps

The XCollar is MRI, X-Ray, and CAT scan compatible.

For research, publications, articles, and videos please visit www.training.xcollar.com or contact us at phone: (805) 684-6568